User Manual A10
Please Read the Manual before Use

Table of contents 1

Product description 2

Parts name 3

Introduction for wireless mode 4

How to charge the battery 4

How to apply the needle cartridge 5

Adjustment of needle length 6

Speed display and adjustment 6

Needle cartridge introduction 7

Reference chart 8

Precaution 9

Warranty 10

Product description 2

The device consists of an electric microneedling pen and disposable needle cartridges. This manual will explain how to use each component in detail.

The device is used to improve the appearance of the skin and reduce signs of aging. It is intended to stimulate the skin with micro-needles for collagen and elastin production. The micro-channels created on surface skin by device also help with the absorption of skin serums.

There are 5 different types of needle cartridges designed for the device. These cartridges can be used for micro-needling treatment and nano-needling treatment.

Parts name 3


Introduction for wireless mode 4

Wireless Model: Dr.pen A10
This model is powered by a built-in battery. It can work wirelessly. The power cable and the adapter are used to charge the device when the battery is low.


How to charge the battery

Connect the device to the adapter with power cable, and plug the adapter into a live AC outlet to charge the device. Charging will be completed within 2 hour.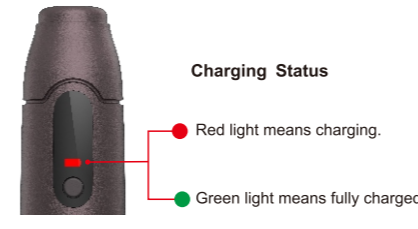


How to apply the needle cartridge 5

Preparation
1. Take out needle cartridge from the sterilized packaging. (Fig1)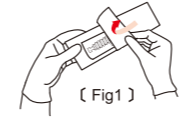

2. Find the "U" shape notch on the cartridge, align the notch with the dot on the bottom of the device, and slightly push the cartridge into the device. (Fig2)

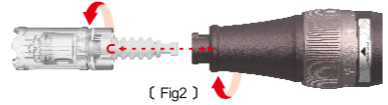3. Lock the needle cartridge to the device by turning it clockwise.

4. Turn the needle cartridge counterclockwise and pull it out when you would like to remove the cartridge.

5. Start the device only if the needle cartridge is assembled correctly. Press and hold the power button for 2-3 seconds to turn on/off the device.

The needle cartridge is designed for single use.
Place the used needle cartridges in a trash container. If the needle cartridge is contaminated inadvertently during installation (i.e. dropped on the floor, package opened/damaged). The needle cartridge may be contaminated. Please discard it and get a new needle cartridge.

Adjustment of needle length 6


Choose suitable needle length according to different treating area and skin problem.

Adjust needle depth by turning the scale adjustment ring. The depth of needle should be adjusted after starting the pen. The scale number aligning with the scale point shows the current needle depth. (Fig3)


Speed display and adjustment

LED light speed display
The blue light turns on when the machine starts.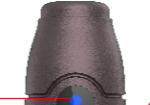

5 Speeds Adjustment
Press the button to change the speed. Turning on one more light means the speed increasing 1 level.

Needle cartridge introduction 7

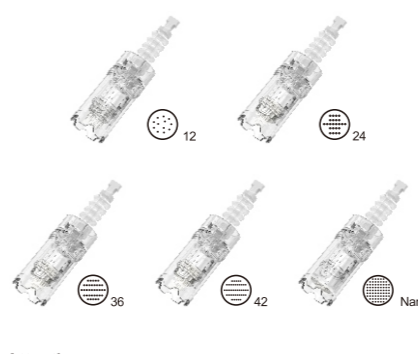Attention
The needle cartridge is designed for Dr.pen A10 only. The needle cartridge is disposable. Please sterilize the needle cartridge with 75% alcohol for few minutes before use. Please dispose the used needle cartridge to the trash container.

Reference chart 8

REGION	NEEDLE DEPTH (MM)
Forehead	0.25 - 0.5
Between the eyebrows	0.25 - 0.5
Nose	0.25
The Area around eyes	0.25
Cheek and Chin	0.5 - 1.0
Surgical scar or wrinkle	2.5

Specification

Product name: Auto-stamp Motorized Meso Machine
Model name: Dr.pen A10
Adapter: Input: 100-240V 50-60HZ Output: 5V-1000mA
Package Weight: 345 g
Package Size: 169*123*56 mm

Item	Quantity
Hand Piece	1 pc
Adapter	1 pc
Manual	1 pc
Power Cable	1 pc
Needle Cartridge	2 pcs

Precaution 9

Please be sure to read the instructions before use and follow as described.

Do not apply it to the person as following:
1. Do not use on areas of severely damaged skin.
2. Do not use on areas with high temperature, dust or humidity.
3. Do not use on areas with high pressure.
4. Do not use on areas with high pressure.
5. Do not use on areas with high pressure.
6. Do not use on areas with high pressure.
7. Do not use on areas with high pressure.

General maintenance
Make sure to inspect the device and adapter before use.
Keep the device with power off. Wipe its surface out with a disinfecting wipes.
Pay attention to clean areas such as crevices, seams, and areas where the needle cartridge attaches to the device.
Keep it away from direct sunlight or high temperature and humidity.
Do not leave the measurement, repair or alter of the device to non-professional staff.
When you use the device which has not been used for a long time, make sure to check if it works properly with safety.

Warranty 10

The warranty period of the device is 12 months. We offer repair service from the purchase date. If any of our products has damages or natural trouble, we provide replacements or repair service. The use of third-party cartridges can cause damage to your device. This could invalidate your warranty.

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